



Lohmann & Rauscher

## Measuring and ordering information




# VENOSAN®

## Measuring Guide

**When?** Where possible, it is recommended to measure in the morning, when legs are rested and swelling is at a minimum.

**How?** Measurements to be taken using a tape measure (cm only) with the patient standing. For Below knee garments, the patient can be measured when sitting. It is recommended that measurements are taken on a bare leg.

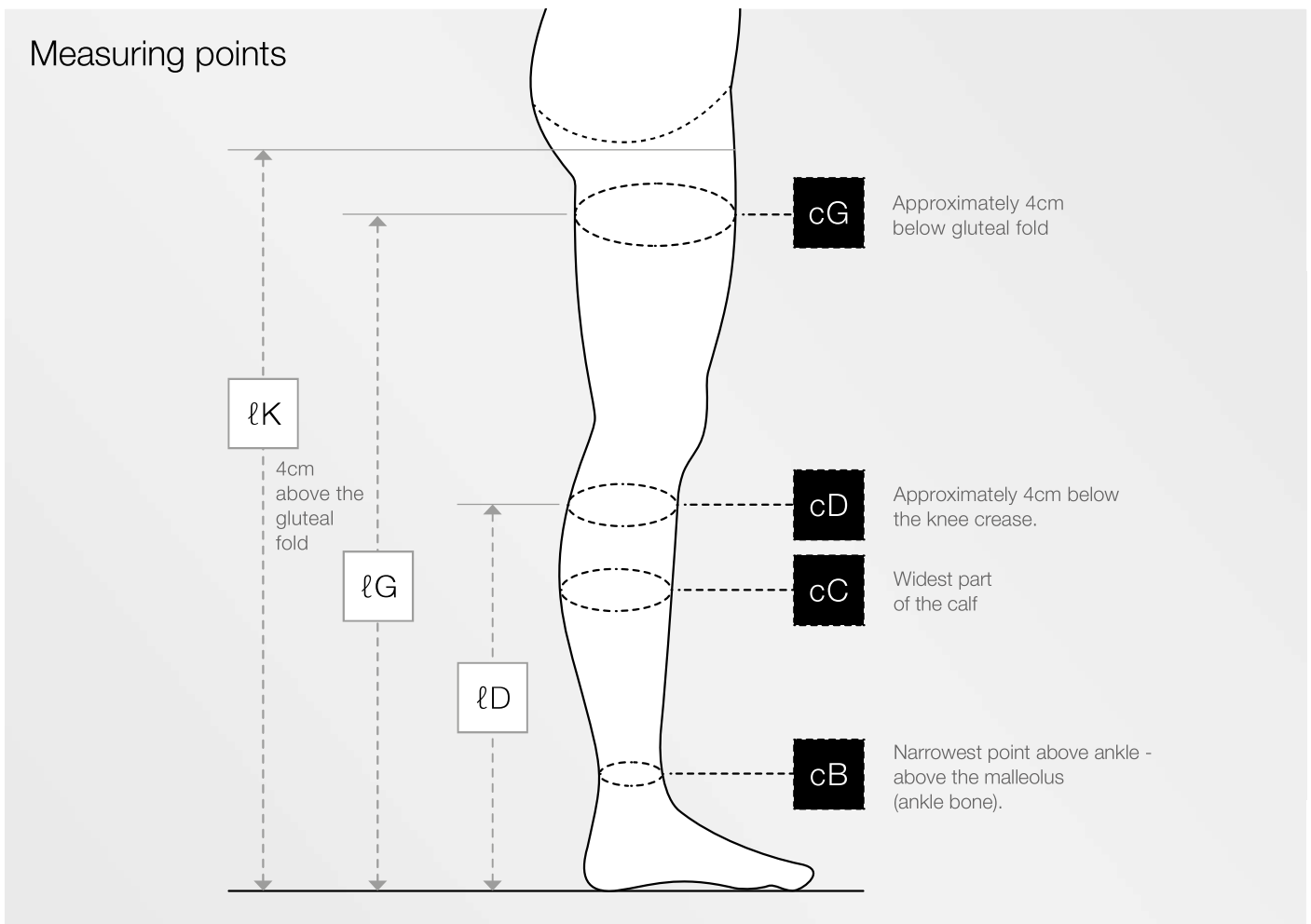

### Which measurements are needed

<p><b>AD</b> Below knee</p> <p><b>ADH</b> Below knee with silicone</p>	<p><b>AGH</b> Thigh length with silicone</p>	<p><b>AGG</b> Thigh length with belt</p>	<p><b>AT</b> Pantyhose</p>	<p><b>ATU</b> Maternity pantyhose</p>
<ul style="list-style-type: none"> <li>■ cB    ■ lD</li> <li>■ cC</li> <li>■ cD</li> </ul>	<ul style="list-style-type: none"> <li>■ cB    ■ lG</li> <li>■ cC</li> <li>■ cD</li> <li>■ cG</li> </ul>	<ul style="list-style-type: none"> <li>■ cB    ■ lK</li> <li>■ cC</li> <li>■ cD</li> <li>■ cG</li> </ul>	<ul style="list-style-type: none"> <li>■ cB    ■ lK</li> <li>■ cC</li> <li>■ cD</li> <li>■ cG</li> </ul>	<ul style="list-style-type: none"> <li>■ cB    ■ lK</li> <li>■ cC</li> <li>■ cD</li> <li>■ cG</li> </ul>

# VENOSAN® Sizing Guide

VENOSAN 3000 VENOSAN 4000 VENOSAN 5000 VENOSAN 7000 VENOSAN 8000

## Circumference

	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)
cB - Ankle	20 – 23	23 – 26	26 – 29	29 – 31
cC - Calf	29 – 39	33 – 42	36 – 45	39 – 47
cD - Below knee	28 – 38	32 – 41	35 – 44	38 – 46
cG - Thigh	48 – 63	54 – 67	58 – 72	63 – 74

## Length

	Length of stocking	Short (cm)	Long (cm)
ℓD - Below knee	AD / ADH	35 – 40	39 – 44
ℓG - Below gluteal fold	AGH	63 – 72	68 – 78
ℓK - Above gluteal fold	AGG / AT / ATU Maternity	67 – 74	74 – 85

## VENOSAN 6000

### Circumference

	X-Small (cm)	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)
cB - Ankle	16 – 19	20 – 22	23 – 25	26 – 28	29 – 31
cC - Calf	27 – 36	30 – 39	33 – 41	36 – 44	39 – 47
cD - Below knee	26 – 35	29 – 38	32 – 40	35 – 43	38 – 46
cG - Thigh	44 – 59	48 – 63	54 – 66	58 – 70	63 – 74

### Length

	Length of stocking	Short (cm)	Long (cm)
ℓD - Below knee	AD / ADH	34 – 39	39 – 44
ℓG - Below gluteal fold	AGH	59 – 67	68 – 78
ℓK - Above gluteal fold	AGG / AT	67 – 74	74 – 85

If patient does not fit into the size range provided, we also have Made To Measure (MTM) available. Contact L&R on 1300 572 869 for more information.




# VENOSAN®

## Measuring and Sizing Guide for Arm Compression Stockings

**When?** It is recommended to measure in the morning as legs are rested and swelling is at a minimum.

**How?** Measurements to be taken using a tape measure (cm only).

### Measuring Points

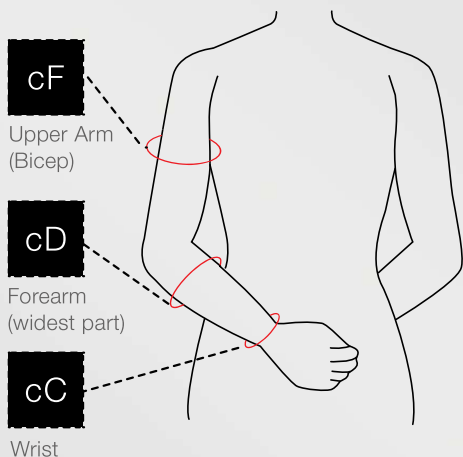

### Sizing Guide

#### Circumference wrist

	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)
cC	14 – 17	17 – 20	20 – 23	23 – 26

#### Circumference forearm

	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)
cD	22 – 26	26 – 30	30 – 34	34 – 38


#### Circumference upper arm

	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)
cF	24 – 30	30 – 36	36 – 42	42 – 48


#### Length

	Short (cm)	Long (cm)
Wrist (C) to Axilla (G)	35 – 45	40 – 50

### Styles available

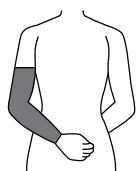

**CG**  
Wrist to Axilla

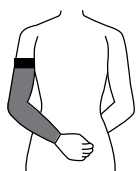

**CG-H**  
Wrist to Axilla  
with silicone band

# VENOSAN® 4000

## CLIMATE EFFECT

Click for VENOSAN 4000 product page

### off-the-shelf range

#### VENOSAN® 4000 CLIMATE EFFECT

CCL  
1

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD Below knee	mexico	long	open	–	SG41001	SG41002	SG41003	SG41004
		long	closed	–	SG41011	SG41012	SG41013	SG41014
		short	open	–	SG41021	SG41022	SG41023	SG41024
		short	closed	–	SG41031	SG41032	SG41033	SG41034
	marokko	long	open	–	SM41001	SM41002	SM41003	SM41004
		long	closed	–	SM41011	SM41012	SM41013	SM41014
		short	open	–	SM41021	SM41022	SM41023	SM41024
		short	closed	–	SM41031	SM41032	SM41033	SM41034
	black	long	open	–	SN41001	SN41002	SN41003	SN41004
		long	closed	–	SN41011	SN41012	SN41013	SN41014
		short	open	–	SN41021	SN41022	SN41023	SN41024
		short	closed	–	SN41031	SN41032	SN41033	SN41034
ADH Below knee with silicone	mexico	long	open	plain top	SG41041	SG41042	SG41043	SG41044
		long	closed	plain top	SG41051	SG41052	SG41053	SG41054
		short	open	plain top	SG41061	SG41062	SG41063	SG41064
		short	closed	plain top	SG41071	SG41072	SG41073	SG41074
	marokko	long	open	plain top	SM41041	SM41042	SM41043	SM41044
		long	closed	plain top	SM41051	SM41052	SM41053	SM41054
		short	open	plain top	SM41061	SM41062	SM41063	SM41064
		short	closed	plain top	SM41071	SM41072	SM41073	SM41074
	black	long	open	plain top	SN41041	SN41042	SN41043	SN41044
		long	closed	plain top	SN41051	SN41052	SN41053	SN41054
		short	open	plain top	SN41061	SN41062	SN41063	SN41064
		short	closed	plain top	SN41071	SN41072	SN41073	SN41074
AGH Thigh length with silicone	mexico	long	closed	lace top	SG41601	SG41602	SG41603	SG41604
		long	open	plain top	SG41611	SG41612	SG41613	SG41614
		long	closed	plain top	SG41621	SG41622	SG41623	SG41624
		short	closed	lace top	SG41631	SG41632	SG41633	SG41634
		short	open	plain top	SG41641	SG41642	SG41643	SG41644
		short	closed	plain top	SG41651	SG41652	SG41653	SG41654
	marokko	long	closed	lace top	SM41601	SM41602	SM41603	SM41604
		long	open	plain top	SM41611	SM41612	SM41613	SM41614
		long	closed	plain top	SM41621	SM41622	SM41623	SM41624
		short	closed	lace top	SM41631	SM41632	SM41633	SM41634
		short	open	plain top	SM41641	SM41642	SM41643	SM41644
		short	closed	plain top	SM41651	SM41652	SM41653	SM41654
	black	long	closed	lace top	SN41601	SN41602	SN41603	SN41604
		long	open	plain top	SN41611	SN41612	SN41613	SN41614
		long	closed	plain top	SN41621	SN41622	SN41623	SN41624
		short	closed	lace top	SN41631	SN41632	SN41633	SN41634
	black	short	open	plain top	SN41641	SN41642	SN41643	SN41644
		short	closed	plain top	SN41651	SN41652	SN41653	SN41654

# VENOSAN® 4000 CLIMATE EFFECT

Click for VENOSAN 4000  
product page

## off-the-shelf range

### VENOSAN® 4000 CLIMATE EFFECT

CCL  
1

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AT Pantyhose	mexico	long	closed	–	SG41221	SG41222	SG41223	SG41224
		short	closed	–	SG41231	SG41232	SG41233	SG41234
	marokko	long	closed	–	SM41221	SM41222	SM41223	SM41224
		short	closed	–	SM41231	SM41232	SM41233	SM41234
	black	long	closed	–	SN41221	SN41222	SN41223	SN41224
		short	closed	–	SN41231	SN41232	SN41233	SN41234
ATU Maternity Pantyhose	mexico	long	closed	–	SG41301	SG41302	SG41303	SG41304
	black	long	closed	–	SN41301	SN41302	SN41303	SN41304

### VENOSAN® 4000 CLIMATE EFFECT

CCL  
2

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD Below knee	mexico	long	open	–	SG42001	SG42002	SG42003	SG42004
		long	closed	–	SG42011	SG42012	SG42013	SG42014
		short	open	–	SG42021	SG42022	SG42023	SG42024
		short	closed	–	SG42031	SG42032	SG42033	SG42034
	marokko	long	open	–	SM42001	SM42002	SM42003	SM42004
		long	closed	–	SM42011	SM42012	SM42013	SM42014
		short	open	–	SM42021	SM42022	SM42023	SM42024
		short	closed	–	SM42031	SM42032	SM42033	SM42034
	black	long	open	–	SN42001	SN42002	SN42003	SN42004
		long	closed	–	SN42011	SN42012	SN42013	SN42014
		short	open	–	SN42021	SN42022	SN42023	SN42024
		short	closed	–	SN42031	SN42032	SN42033	SN42034
ADH Below knee with silicone	mexico	long	open	plain top	SG42041	SG42042	SG42043	SG42044
		long	closed	plain top	SG42051	SG42052	SG42053	SG42054
		short	open	plain top	SG42061	SG42062	SG42063	SG42064
		short	closed	plain top	SG42071	SG42072	SG42073	SG42074
	marokko	long	open	plain top	SM42041	SM42042	SM42043	SM42044
		long	closed	plain top	SM42051	SM42052	SM42053	SM42054
		short	open	plain top	SM42061	SM42062	SM42063	SM42064
		short	closed	plain top	SM42071	SM42072	SM42073	SM42074
	black	long	open	plain top	SN42041	SN42042	SN42043	SN42044
		long	closed	plain top	SN42051	SN42052	SN42053	SN42054
		short	open	plain top	SN42061	SN42062	SN42063	SN42064
		short	closed	plain top	SN42071	SN42072	SN42073	SN42074

# VENOSAN® 4000 CLIMATE EFFECT

## off-the-shelf range

CCL  
2

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AGH Thigh length with silicone	mexico	long	closed	lace top	SG42601	SG42602	SG42603	SG42604
		long	open	plain top	SG42611	SG42612	SG42613	SG42614
		long	closed	plain top	SG42621	SG42622	SG42623	SG42624
		short	closed	lace top	SG42631	SG42632	SG42633	SG42634
		short	open	plain top	SG42641	SG42642	SG42643	SG42644
		short	closed	plain top	SG42651	SG42652	SG42653	SG42654
	marokko	long	closed	lace top	SM42601	SM42602	SM42603	SM42604
		long	open	plain top	SM42611	SM42612	SM42613	SM42614
		long	closed	plain top	SM42621	SM42622	SM42623	SM42624
		short	closed	lace top	SM42631	SM42632	SM42633	SM42634
		short	open	plain top	SM42641	SM42642	SM42643	SM42644
		short	closed	plain top	SM42651	SM42652	SM42653	SM42654
	black	long	closed	lace top	SN42601	SN42602	SN42603	SN42604
		long	open	plain top	SN42611	SN42612	SN42613	SN42614
		long	closed	plain top	SN42621	SN42622	SN42623	SN42624
		short	closed	lace top	SN42631	SN42632	SN42633	SN42634
		short	open	plain top	SN42641	SN42642	SN42643	SN42644
		short	closed	plain top	SN42651	SN42652	SN42653	SN42654
AGG (left) Thigh length with belt	mexico	long	open	-	SG42151	SG42152	SG42153	SG42154
AGG (right) Thigh length with belt	mexico	long	open	-	SG42171	SG42172	SG42173	SG42174
AT Pantyhose	mexico	long	closed	-	SG42221	SG42222	SG42223	SG42224
		short	closed	-	SG42231	SG42232	SG42233	SG42234
	marokko	long	closed	-	SM42221	SM42222	SM42223	SM42224
		short	closed	-	SM42231	SM42232	SM42233	SM42234
	black	long	closed	-	SN42221	SN42222	SN42223	SN42224
		short	closed	-	SN42231	SN42232	SN42233	SN42234
ATU Maternity Pantyhose	mexico	long	closed	-	SG42301	SG42302	SG42303	SG42304
		short	closed	-	SG42311	SG42312	SG42313	SG42314
	black	long	closed	-	SN42301	SN42302	SN42303	SN42304
		short	closed	-	SN42311	SN42312	SN42313	SN42314

## made-to-measure

### VENOSAN® 4000 CLIMATE EFFECT

CCL  
1

CCL  
2

Style	Colour	Toe	CCL 1	CCL 2
AD	CCL 1 CCL 2 mexico, black	open/closed	•	•
ADH		open/closed	•	•
AG		open/closed	•	•
AGH		open/closed	•	•
AGG		open/closed	•	•
AT		open/closed	•	•
ATU Maternity		open/closed	•	•

• = available

# VENOSAN® 5000

## THE MAGIC OF SILVER

### off-the-shelf range

#### VENOSAN® 5000

##### THE MAGIC OF SILVER

CCL  
1

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD Below knee	mexico	long	open	–	SG45001	SG45002	SG45003	SG45004
		long	closed	–	SG45011	SG45012	SG45013	SG45014
	marokko	long	open	–	SM45001	SM45002	SM45003	SM45004
		long	closed	–	SM45011	SM45012	SM45013	SM45014
	black	long	open	–	SN45001	SN45002	SN45003	SN45004
long		closed	–	SN45011	SN45012	SN45013	SN45014	
AGH Thigh length with silicone	mexico	long	closed	lace top	SG45601	SG45602	SG45603	SG45604
		long	open	plain top	SG45611	SG45612	SG45613	SG45614
	marokko	long	closed	lace top	SM45601	SM45602	SM45603	SM45604
		long	open	plain top	SM45611	SM45612	SM45613	SM45614
	black	long	closed	lace top	SN45601	SN45602	SN45603	SN45604
		long	open	plain top	SN45611	SN45612	SN45613	SN45614
AGG (left) Thigh length with belt	mexico	long	open	–	SG45151	SG45152	SG45153	SG45154
AGG (right) Thigh length with belt	mexico	long	open	–	SG45171	SG45172	SG45173	SG45174
AT Pantyhose	mexico	long	closed	–	SG45221	SG45222	SG45223	SG45224
	marokko	long	closed	–	SM45221	SM45222	SM45223	SM45224
	black	long	closed	–	SN45221	SN45222	SN45223	SN45224

#### VENOSAN® 5000

##### THE MAGIC OF SILVER

CCL  
2

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD Below knee	mexico	long	open	–	SG46001	SG46002	SG46003	SG46004
		long	closed	–	SG46011	SG46012	SG46013	SG46014
	marokko	long	open	–	SM46001	SM46002	SM46003	SM46004
		long	closed	–	SM46011	SM46012	SM46013	SM46014
	black	long	open	–	SN46001	SN46002	SN46003	SN46004
long		closed	–	SN46011	SN46012	SN46013	SN46014	
AGH Thigh length with silicone	mexico	long	open	plain top	SG46611	SG46612	SG46613	SG46614
		long	closed	plain top	SG46621	SG46622	SG46623	SG46624
	marokko	long	open	plain top	SM46611	SM46612	SM46613	SM46614
		long	closed	plain top	SM46621	SM46622	SM46623	SM46624
	black	long	open	plain top	SN46611	SN46612	SN46613	SN46614
		long	closed	plain top	SN46621	SN46622	SN46623	SN46624
AGG (left) Thigh length with belt	mexico	long	open	–	SG46151	SG46152	SG46153	SG46154
AGG (right) Thigh length with belt	mexico	long	open	–	SG46171	SG46172	SG46173	SG46174
AT Pantyhose	mexico	long	closed	–	SG46221	SG46222	SG46223	SG46224
	marokko	long	closed	–	SM46221	SM46222	SM46223	SM46224
	black	long	closed	–	SN46221	SN46222	SN46223	SN46224



Click for VENOSAN 5000 product page

# VENOSAN® 5000

## THE MAGIC OF SILVER

### off-the-shelf range

#### VENOSAN® 5000

THE MAGIC OF SILVER

CCL  
3

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD Below knee	mexico	long	open	–	SG47001	SG47002	SG47003	SG47004
		long	closed	–	SG47011	SG47012	SG47013	SG47014
	marokko	long	open	–	SM47001	SM47002	SM47003	SM47004
		long	closed	–	SM47011	SM47012	SM47013	SM47014
	black	long	open	–	SN47001	SN47002	SN47003	SN47004
		long	closed	–	SN47011	SN47012	SN47013	SN47014

### made-to-measure

#### VENOSAN® 5000

THE MAGIC OF SILVER

CCL  
1

CCL  
2

CCL  
3

Style	Colour	Toe	CCL 1	CCL 2	CCL 3
AD		open/closed	•	•	•
ADH	CCL 1	open/closed	•	•	•
AG	CCL 2	open/closed	•	•	•
AGH	mexico, black	open/closed	•	•	•
AGG		open/closed	•	•	•
AT	CCL 3	open/closed	•	•	•
ATU Maternity	mexico	open/closed	•	•	•

• = available

# VENOSAN® 6000 CLASSIC

## off-the-shelf range

### VENOSAN® 6000 CLASSIC

CCL  
1

Style	Colour	Length	Toe	Silicone Band	REF				
					XS	S	M	L	XL
AD Below knee	beige	long	open	–	CG61000	CG61001	CG61002	CG61003	CG61004
		long	closed	–	CG61010	CG61011	CG61012	CG61013	CG61014
		short	open	–	CG61020	CG61021	CG61022	CG61023	CG61024
		short	closed	–	CG61030	CG61031	CG61032	CG61033	CG61034
	black	long	open	–	CN61000	CN61001	CN61002	CN61003	CN61004
		long	closed	–	CN61010	CN61011	CN61012	CN61013	CN61014
		short	open	–	CN61020	CN61021	CN61022	CN61023	CN61024
		short	closed	–	CN61030	CN61031	CN61032	CN61033	CN61034
AGH Thigh length with silicone	beige	long	open	plain top	CG61600	CG61601	CG61602	CG61603	CG61604
		long	closed	plain top	CG61610	CG61611	CG61612	CG61613	CG61614
		short	open	plain top	CG61620	CG61621	CG61622	CG61623	CG61624
		short	closed	plain top	CG61630	CG61631	CG61632	CG61633	CG61634
	black	long	open	plain top	CN61600	CN61601	CN61602	CN61603	CN61604
		long	closed	plain top	CN61610	CN61611	CN61612	CN61613	CN61614
		short	open	plain top	CN61620	CN61621	CN61622	CN61623	CN61624
		short	closed	plain top	CN61630	CN61631	CN61632	CN61633	CN61634
AT Pantyhose	beige	long	closed	–	CG61210	CG61211	CG61212	CG61213	CG61214
		short	closed	–	CG61230	CG61231	CG61232	CG61233	CG61234
	black	long	closed	–	CN61210	CN61211	CN61212	CN61213	CN61214
		short	closed	–	CN61230	CN61231	CN61232	CN61233	CN61234

### VENOSAN® 6000 CLASSIC

CCL  
2

Style	Colour	Length	Toe	Silicone Band	REF				
					XS	S	M	L	XL
AD Below knee	beige	long	open	–	CG62000	CG62001	CG62002	CG62003	CG62004
		long	closed	–	CG62010	CG62011	CG62012	CG62013	CG62014
		short	open	–	CG62020	CG62021	CG62022	CG62023	CG62024
		short	closed	–	CG62030	CG62031	CG62032	CG62033	CG62034
	black	long	open	–	CN62000	CN62001	CN62002	CN62003	CN62004
		long	closed	–	CN62010	CN62011	CN62012	CN62013	CN62014
		short	open	–	CN62020	CN62021	CN62022	CN62023	CN62024
		short	closed	–	–	CN62231	CN62232	CN62233	CN62234

# VENOSAN® 6000 CLASSIC

off-the-shelf range

## VENOSAN® 6000 CLASSIC

CCL  
2

Style	Colour	Length	Toe	Silicone Band	REF				
					XS	S	M	L	XL
AGH Thigh length with silicone	beige	long	open	plain top	CG62600	CG62601	CG62602	CG62603	CG62604
		long	closed	plain top	–	CG62611	CG62612	CG62613	CG62614
		short	open	plain top	CG62620	CG62621	CG62622	CG62623	CG62624
		short	closed	plain top	–	CG62631	CG62632	CG62633	CG62634
	black	long	open	plain top	CN62600	CN62601	CN62602	CN62603	CN62604
		long	closed	plain top	–	CN62611	CN62612	CN62613	CN62614
		short	open	plain top	CN62620	CN62621	CN62622	CN62623	CN62624
		short	closed	plain top	–	CN62631	CN62632	CN62633	CN62634
AGG (left) Thigh length with belt	beige	long	open	–	CG62150	CG62151	CG62152	CG62153	CG62154
		short	open	–	CG62160	CG62161	CG62162	CG62163	CG62164
AGG (right) Thigh length with belt	beige	long	open	–	CG62170	CG62171	CG62172	CG62173	CG62174
		short	open	–	CG62180	CG62181	CG62182	CG62183	CG62184
AT Pantyhose	beige	long	closed	–	CG62210	CG62211	CG62212	CG62213	CG62214
		short	closed	–	CG62230	CG62231	CG62232	CG62233	CG62234
	black	long	closed	–	CN62210	CN62211	CN62212	CN62213	CN62214
		short	closed	–	CN62230	CN62231	CN62232	CN62233	CN62234

## VENOSAN® 6000 CLASSIC

CCL  
3

Style	Colour	Length	Toe	Silicone Band	REF				
					XS	S	M	L	XL
AD Below knee	beige	long	open	–	CG63000	CG63001	CG63002	CG63003	CG63004
		short	open	–	CG63020	CG63021	CG63022	CG63023	CG63024
AGH Thigh length with silicone	beige	long	open	plain top	CG63600	CG63601	CG63602	CG63603	CG63604
		short	open	plain top	CG63620	CG63621	CG63622	CG63623	CG63624
AT Pantyhose	beige	long	open	–	CG63200	CG63201	CG63202	CG63203	CG63204
		short	open	–	CG63020	CG63021	CG63022	CG63023	CG63024

# VENOSAN® 7000

## THE MAGIC OF SILVER

### off-the-shelf range

#### VENOSAN® 7000 NATURAL TOUCH

CCL  
1

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD Below knee	beige	long	open	–	SG71001	SG71002	SG71003	SG71004
		long	closed	–	SG71011	SG71012	SG71013	SG71014
		short	open	–	SG71021	SG71022	SG71023	SG71024
		short	closed	–	SG71031	SG71032	SG71033	SG71034

#### VENOSAN® 7000 NATURAL TOUCH

CCL  
2

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD Below knee	beige	long	open	–	SG72001	SG72002	SG72003	SG72004
		long	closed	–	SG72011	SG72012	SG72013	SG72014
		short	open	–	SG72021	SG72022	SG72023	SG72024
		short	closed	–	SG72031	SG72032	SG72033	SG72034
	black	long	open	–	SN72001	SN72002	SN72003	SN72004
		long	closed	–	SN72011	SN72012	SN72013	SN72014
		short	open	–	SN72021	SN72022	SN72023	SN72024
		short	closed	–	SN72031	SN72032	SN72033	SN72034
ADH Below knee with silicone	beige	long	open	plain top	SG72041	SG72042	SG72043	SG72044
		long	closed	plain top	SG72051	SG72052	SG72053	SG72054
		short	open	plain top	SG72061	SG72062	SG72063	SG72064
		short	closed	plain top	SG72071	SG72072	SG72073	SG72074
	black	long	open	plain top	SN72041	SN72042	SN72043	SN72044
		long	closed	plain top	SN72051	SN72052	SN72053	SN72054
		short	open	plain top	SN72061	SN72062	SN72063	SN72064
		short	closed	plain top	SN72071	SN72072	SN72073	SN72074
AGH Thigh length with silicone	beige	long	open	plain top	SG72601	SG72602	SG72603	SG72604
		long	closed	plain top	SG72611	SG72612	SG72613	SG72614
		short	open	plain top	SG72621	SG72622	SG72623	SG72624
		short	closed	plain top	SG72631	SG72632	SG72633	SG72634
	black	long	open	plain top	SN72601	SN72602	SN72603	SN72604
		long	closed	plain top	SN72611	SN72612	SN72613	SN72614
		short	open	plain top	SN72621	SN72622	SN72623	SN72624
		short	closed	plain top	SN72631	SN72632	SN72633	SN72634
AGG (left) Thigh length with belt	beige	long	open	–	SG72151	SG72152	SG72153	SG72154
		short	open	–	SG72161	SG72162	SG72163	SG72164
AGG (right) Thigh length with belt	beige	long	open	–	SG72171	SG72172	SG72173	SG72174
		short	open	–	SG72181	SG72182	SG72183	SG72184

# VENOSAN® 7000

## THE MAGIC OF SILVER

### off-the-shelf range

#### VENOSAN® 7000 NATURAL TOUCH

CCL  
3

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD Below knee	beige	long	open	–	SG73001	SG73002	SG73003	SG73004
		long	closed	–	SG73011	SG73012	SG73013	SG73014
		short	open	–	SG73021	SG73022	SG73023	SG73024
		short	closed	–	SG73031	SG73032	SG73033	SG73034
	black	long	open	–	SN73001	SN73002	SN73003	SN73004
		long	closed	–	SN73011	SN73012	SN73013	SN73014
		short	open	–	SN73021	SN73022	SN73023	SN73024
		short	closed	–	SN73031	SN73032	SN73033	SN73034
ADH Below knee with silicone	beige	long	open	plain top	SG73041	SG73042	SG73043	SG73044
		long	closed	plain top	SG73051	SG73052	SG73053	SG73054
		short	open	plain top	SG73061	SG73062	SG73063	SG73064
		short	closed	plain top	SG73071	SG73072	SG73073	SG73074
	black	long	open	plain top	SN73041	SN73042	SN73043	SN73044
		long	closed	plain top	SN73051	SN73052	SN73053	SN73054
		short	open	plain top	SN73061	SN73062	SN73063	SN73064
		short	closed	plain top	SN73071	SN73072	SN73073	SN73074
AGH Thigh length with silicone	beige	long	open	plain top	SG73601	SG73602	SG73603	SG73604
		long	closed	plain top	SG73611	SG73612	SG73613	SG73614
		short	open	plain top	SG73621	SG73622	SG73623	SG73624
		short	closed	plain top	SG73631	SG73632	SG73633	SG73634
	black	long	open	plain top	SN73601	SN73602	SN73603	SN73604
		long	closed	plain top	SN73611	SN73612	SN73613	SN73614
		short	open	plain top	SN73621	SN73622	SN73623	SN73624
		short	closed	plain top	SN73631	SN73632	SN73633	SN73634

### made-to-measure

#### VENOSAN® 7000 NATURAL TOUCH

CCL  
2

CCL  
3

Style	Colour	Toe	CCL 2	CCL 3
AD	beige, black	open/closed	•	•
ADH		open/closed	•	•
AG		open/closed	•	•
AGH		open/closed	•	•
AGG		open/closed	•	•
AT		open/closed	•	•
ATH		open/closed	•	•

• = available

# VENOSAN® 8000 ULCERFIT COMPRESSION STOCKING SYSTEM

## off-the-shelf range

### VENOSAN® 8000 ULCERFIT COMPRESSION STOCKING SYSTEM

CCL  
2

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD (Set*) Below knee	beige / white	long 37 – 42cm	understocking: closed	–	SG82001	SG82002	SG82003	SG82004
		short 35 – 40cm	overstocking: open	–	SG82021	SG82022	SG82023	SG82024

\* Set: consisting of one (beige) overstocking and two (white) understockings

### VENOSAN® 8000 ULCERFIT COMPRESSION STOCKING SYSTEM

CCL  
3

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD (Set*) Below knee	beige / white	long 37 – 42cm	understocking: closed	–	SG83001	SG83002	SG83003	SG83004
		short 35 – 40cm	overstocking: open	–	SG83021	SG83022	SG83023	SG83024

\* Set: consisting of one (beige) overstocking and two (white) understockings

### VENOSAN® 8000 ULCERFIT LINERS

CCL  
3

Style	Colour	Length	Toe	Silicone Band	REF			
					S	M	L	XL
AD (Pair) Below knee	white	35 – 42cm	closed	–	SG80001	SG80002	SG80003	SG80004

# VENOSAN® 4000 Armsleeves

## CLIMATE EFFECT

Click for VENOSAN 4000 Armsleeves product page

### off-the-shelf range

#### VENOSAN® 4000 Armsleeves

##### CLIMATE EFFECT

CCL  
2

Style	Colour	Length	Silicone Band	REF			
				S	M	L	XL
<b>CG</b> Wrist to axilla	mexico	long	–	SG42501	SG42502	SG42503	SG42504

### made-to-measure

#### VENOSAN® 4000 Armsleeves

##### CLIMATE EFFECT

Style	Colour	With or without hand piece	4002 CCL 2
<b>CG</b>	beige,	without hand	•
<b>CG-H</b>	black	without hand	•

• = available

# VENOSAN® 7000 Armsleeves

## NATURAL TOUCH

### off-the-shelf range

#### VENOSAN® 7000 Armsleeves

##### NATURAL TOUCH

CCL  
2

Style	Colour	Length	With or without hand piece	Silicone Band	REF			
					S	M	L	XL
<b>CG</b> Wrist to axilla	beige	long	without hand	–	SG72501	SG72502	SG72503	SG72504
		short	without hand	–	SG72511	SG72512	SG72513	SG72514
<b>CG-H</b> Wrist to axilla with silicone	beige	long	without hand	plain top	SG72521	SG72522	SG72523	SG72524
		short	without hand	plain top	SG72531	SG72532	SG72533	SG72534
<b>ZG</b> MCP to axilla	beige	long	with hand	–	SG72541	SG72542	SG72543	SG72544
		short	with hand	–	SG72551	SG72552	SG72553	SG72554
<b>ZG-H</b> MCP to Axilla with silicone	beige	long	with hand	plain top	SG72561	SG72562	SG72563	SG72564
		short	with hand	plain top	SG72571	SG72572	SG72573	SG72574

### made-to-measure

#### VENOSAN® 7000 Armsleeves

##### NATURAL TOUCH

Style	Colour	With or without hand piece	7002 CCL 2	7003 CCL 3
<b>CG</b>	beige, black	without hand	•	•
<b>CG-H</b>		without hand	•	•
<b>CH</b>		without hand	•	•
<b>ZG</b>		with hand	•	•
<b>ZG-H</b>		with hand	•	•
<b>ZH</b>		with hand	•	•

• = available



Click for VENOSAN LEGLINE 20 product page


# VENOSAN® LEGLINE 20

## ELEGANT LUXURY

### off-the-shelf range

#### Sizing Guide

Circumference	X-Small (cm)	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)
cB - Ankle	20 – 21	22 – 23	24 – 25	26 – 27	28 – 29
cC - Calf	31 – 33	34 – 36	37 – 39	40 – 42	43 – 45
cD - Below knee	29 – 31	32 – 34	35 – 37	38 – 40	41 – 43
cF - Mid thigh	42 – 44	45 – 47	48 – 52	51 – 53	54 – 60

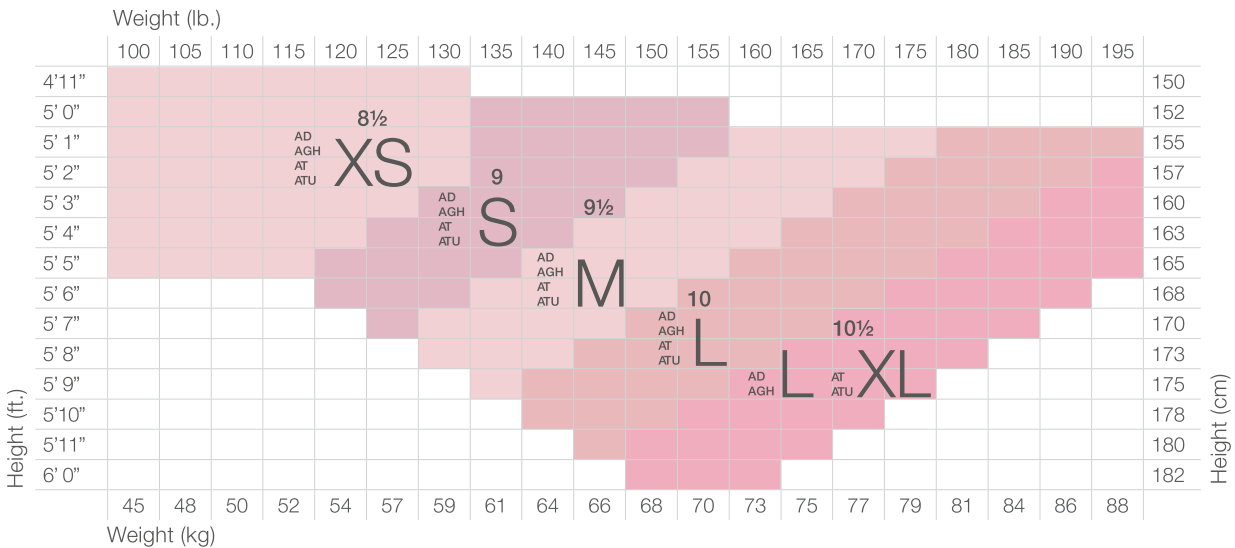

#### Ordering Information

### VENOSAN® LEGLINE 20

#### ELEGANT LUXURY

20mmHg

Style	Colour	Toe	Silicone Band	REF				
				XS	S	M	L	XL
AD Below knee	nude	closed	–	VL303NU	VL304NU	VL305NU	VL306NU	
	sahara	closed	–	VL303SA	VL304SA	VL305SA	VL306SA	
	black	closed	–	VL303BL	VL304BL	VL305BL	VL306BL	
AGH Thigh length with silicone	nude	closed	lace top	VL323NU	VL324NU	VL325NU	VL326NU	
	sahara	closed	lace top	VL323SA	VL324SA	VL325SA	VL326SA	
	black	closed	lace top	VL323BL	VL324BL	VL325BL	VL326BL	
AT Pantyhose	nude	closed	–	VL333NU	VL334NU	VL335NU	VL336NU	VL337NU
	sahara	closed	–	VL333SA	VL334SA	VL335SA	VL336SA	VL337SA
	black	closed	–	VL333BL	VL334BL	VL335BL	VL336BL	VL337BL
ATU Maternity Pantyhose	sahara	closed	–	VL343SA	VL344SA	VL345SA	VL346SA	VL347SA
	black	closed	–	VL343BL	VL344BL	VL345BL	VL346BL	VL347BL

Measuring and ordering information

# VENOSAN® LEGLINE 30


## ELEGANT LUXURY

Click for VENOSAN LEGLINE 30 product page

### off-the-shelf range

#### Sizing Guide

Circumference	X-Small (cm)	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)
cB - Ankle	20 – 21	22 – 23	24 – 25	26 – 27	28 – 29
cC - Calf	31 – 33	34 – 36	37 – 39	40 – 42	43 – 45
cD - Below knee	29 – 31	32 – 34	35 – 37	38 – 40	41 – 43
cF - Mid thigh	42 – 44	45 – 47	48 – 52	51 – 53	54 – 60

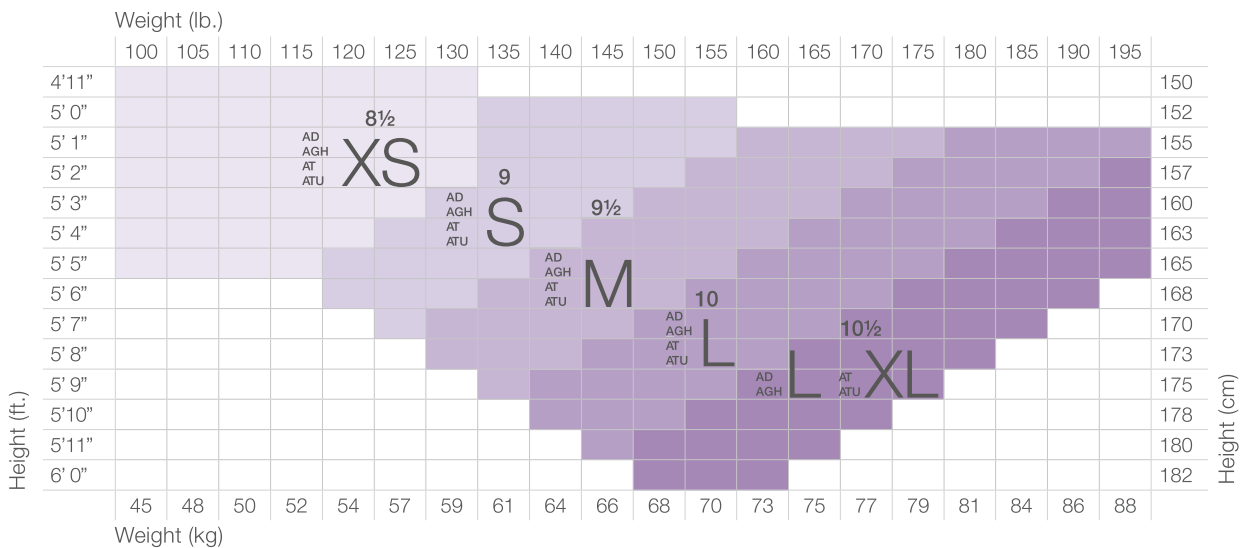

#### Ordering Information

### VENOSAN® LEGLINE 30

#### ELEGANT LUXURY

30mmHg

Style	Colour	Toe	Silicone Band	REF				
				XS	S	M	L	XL
AD Below knee	nude	closed	–	VL403NU	VL404NU	VL405NU	VL406NU	
	sahara	closed	–	VL403SA	VL404SA	VL405SA	VL406SA	
	black	closed	–	VL403BL	VL404BL	VL405BL	VL406BL	
AGH Thigh length with silicone	nude	closed	lace top	VL423NU	VL424NU	VL425NU	VL426NU	
	sahara	closed	lace top	VL423SA	VL424SA	VL425SA	VL426SA	
	black	closed	lace top	VL423BL	VL424BL	VL425BL	VL426BL	
AT Pantyhose	nude	closed	–	VL433NU	VL434NU	VL435NU	VL436NU	VL437NU
	sahara	closed	–	VL433SA	VL434SA	VL435SA	VL436SA	VL437SA
	black	closed	–	VL433BL	VL434BL	VL435BL	VL436BL	VL437BL

# VENOSAN® MICROFIBERLINE

## STYLISH COMFORT, FASHIONABLY SOFT

### off-the-shelf range

#### Sizing Guide

##### VENOSAN® MicroFiberLine Gentlemen

Circumference	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)	XX-Large (cm)
cB - Ankle	19 – 21.5	21.5 – 24	24 – 26.5	26.5 – 32	32 – 36
cC - Calf	28 – 38	33 – 43.5	35.5 – 46	38.5 – 51.5	41 – 53

##### VENOSAN® MicroFiberLine Lady

Circumference	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)	XX-Large (cm)
cB - Ankle	16.5 – 20.5	20.5 – 24.5	24.5 – 28	28 – 32	32 – 36
cC - Calf	26.5 – 37	29 – 39.5	32 – 43	34.5 – 47	41 – 53

#### Ordering Information

##### VENOSAN® MicroFiberLine Gentlemen

Style	Compression	Colour	Toe	REF			
				S	M	L	XL
AD Below knee	15 – 20mmHg	black	closed	MC79002	MC79003	MC79004	MC79005
	20 – 30mmHg	black	closed	MC89002	MC89003	MC89004	MC89005

##### VENOSAN® MicroFiberLine Lady

Style	Compression	Colour	Toe	REF			
				S	M	L	XL
AD Below knee	15 – 20mmHg	black	closed	MC79221	MC79222	MC79223	MC79224
		beige	closed	MC79205	MC79206	MC79207	MC79208

Click for VENOSAN  
MICROFIBERLINE product page

# VENOSAN® SILVERLINE

## EVERYDAY COMFORT

### off-the-shelf range

#### Sizing Guide

##### VENOSAN® Silverline Gentlemen

Circumference	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)	XX-Large (cm)
cB - Ankle	19 – 21.5	21.5 – 24	24 – 26.5	26.5 – 32	32 – 36
cC - Calf	28 – 38	33 – 43.5	35.5 – 46	38.5 – 51.5	41 – 53

##### VENOSAN® Silverline Lady

Circumference	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)	XX-Large (cm)
cB - Ankle	16.5 – 20.5	20.5 – 24.5	24.5 – 28	28 – 32	32 – 36
cC - Calf	26.5 – 37	29 – 39.5	32 – 43	34.5 – 47	41 – 53

#### Ordering Information

##### VENOSAN® Silverline Gentlemen

Style	Compression	Colour	Toe	REF				
				S	M	L	XL	XXL
AD Below knee	20-30mmHg	black	closed	VS49405	VS49406	VS49407	VS49408	VS49409

##### VENOSAN® Silverline Lady

Style	Compression	Colour		Toe	REF			
		black	beige		S	M	L	XL
AD Below knee	20-30mmHg	black	closed	VS49505	VS49506	VS49507	VS49508	VS49509
		beige	closed	VS49511	VS49512	VS49513	VS49514	VS49515

Click for VENOSAN SILVERLINE product page

# VENOSAN® ATHLETIC PERFORMANCE SUPPORT

## off-the-shelf range

### Sizing Guide

Circumference	Small (cm)	Medium (cm)	Large (cm)	X-Large (cm)
cB - Ankle	18 – 21	22 – 25	25 – 29	29 – 33
cC - Calf	29 – 38	30 – 42	32 – 46	34 – 50

### Ordering Information

Style	Compression	Colour	Toe	REF			
				S	M	L	XL
AD Below knee	20 – 30mmHg	white	closed	A59202W	A59203W	A59204W	A59205W
		black	closed	A59202B	A59203B	A59204B	A59205B
		pink	closed	A59202F	A59203F	A59204F	A59205F
		purple	closed	A59202P	A59203P	A59204P	A59205P
		blue	closed	A59202S	A59203S	A59204S	A59205S

[Click for VENOSAN ATHLETIC product page](#)