


# Adva-co®

## Compression bandage

### Product description

Adva-co® is a lightweight long-stretch high compression bandage. It will provide sustained compression at optimum healing levels for leg ulcers. The gold standard is 40mm Hg sub-bandage pressure at the ankle and 20mm Hg at the knee (graduated compression).

Adva-co® is used with a sub-bandage wadding layer (e.g. Advasoft) making it a cost effective treatment for the management of venous insufficiency.

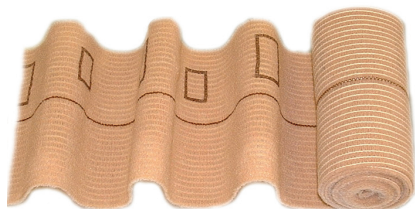

### Use

The bandage has two rectangular pressure indicators to help ensure the correct pressure is achieved. A rectangle elongated to a square indicates the correct pressure (as below).



For comfort and distribution of pressure use Advasoft® (p21) wadding and secure bandage with Advaflex® tabs (supplied). Protect open wounds with an appropriate Advancis dressing of choice.

High compression bandages must not be used on arterial ulcers. See the Advancis guide to leg ulcers for more information.

An educational guide to leg ulceration is available to download on our website [www.advancis.co.uk](http://www.advancis.co.uk) or free copies can be requested by emailing [info@advancis.co.uk](mailto:info@advancis.co.uk).

### Features

- ✓ **Lightweight breathable bandage**
- ✓ **Two sizes of the rectangular pressure indicator for effective management of compression therapy**
- ✓ **Thin material to promote greater patient compliance**
- ✓ **Centre line to assist a 50% overlap of the bandage**
- ✓ **Soft finish for improved comfort**
- ✓ **Accurate reliable guaranteed compression**
- ✓ **Latex Free**

### Indications

- ✓ **For the management of venous leg ulcers and associated conditions where patients have poor venous return and where compression therapy is indicated**

Advaflex® is a non-adhesive system for the simple and secure fixation of bandages. The product consists of individual micro-hook tabs supplied on a reel. The individual tabs can be easily separated from each other and applied to a wide range of bandages to secure the ends. Available in boxes of 200.



Product	Size	Stock code	No. per box	PIP code
Adva-co®	10cm x 3.5m	KN3627	12	301-1681
Advaflex®	15 x 54mm	BG3717	200	n/a

# Advasoft®

## Sub-compression wadding bandage

### Product description

A soft comfortable and conformable wadding bandage with excellent moisture handling and absorbency features. The product combines a high level of absorbency, making it comfortable to wear, with a high degree of pressure distribution and resistance to the forces exerted under compression.

Advasoft® is easy to apply due to its conformability, good level of self adherence and easy tear properties. A combination of viscose (cellulose) and polyester fibres provide Advasoft® with an advantageous wet strength quality.

Due to the manufacturing process the fibres used in Advasoft® have a high degree of cleanliness and are very suitable for skin contact.

### Features

- ✓ **Soft and conformable**
- ✓ **Moisture absorption**
- ✓ **Pressure distribution**
- ✓ **Easy tear**
- ✓ **Wet strength**

### Indications

- ✓ **Use for the distribution of pressure under compression bandages and/or the protection of bony prominences, can also be used as an orthopaedic under cast padding**



Product	Size	Stock code	Unit of sale	PIP code
Advasoft®	10cm x 3.5m	CR3661	6	301-1657