

# Size Chart - VENOSAN 4000, 5000, 7000 & 8000 series

## Measurement Guidelines

In order to work effectively graduated compression hosiery must be the correct size and apply the right amount of pressure to the leg. It is essential that accurate measurements are taken.

- All measurements should be taken on the bare leg, in centimetres, whilst standing (if possible).
- Ensure that the tape measure is snug but not too tight.
- Measurements should be taken when the leg is rested and not swollen


Some of these products have Maximum leg lengths for each style of hose. Please refer to measurements guidelines and chart below for instructions.

### Below Knee,

- The circumference of the ankle **cB**
- The circumference of the calf **cC**
- the circumference of the leg just below the knee **cD**
- the length of the leg from the base of the heel to just below the knee **ID**

### Thigh length, Tights (Pantyhose) & Maternity Tights (Maternity Pantyhose)

- The circumference of the ankle **cB**
- The circumference of the calf **cC**
- the circumference of the leg just below the knee **cD**
- the circumference of the leg just below the knee **cG**
- the inside leg **IG** (for AGG/AT/ATU up to the groin **IK**)

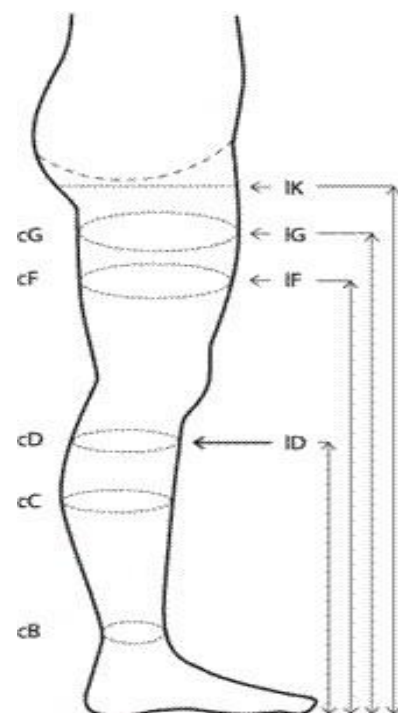

## Circumference

Circumference (cm)	Small	Medium	Large	Extra Large
cB Ankle	20-23	23-26	26-29	29-31 cm
cC Mid Calf	29-39	33-42	36-45	39-47 cm
cD Below Knee	28-38	32-41	35-44	38-46 cm
cG Thigh	48-63	54-67	58-72	63-74 cm

## Length

Below Knee Hosiery	Short Long	35-40 cm 37-42 cm
Thigh Length Hosiery	Short Long	63-72 cm 68-78 cm
Thigh length, with waist support Tights (Pantyhose) Maternity Tights	Short Long	67-74 cm 74-85 cm